MEDICAL TRADING
ITALIAN MANUFACTURER
OF AUTOCLAVE
FOR STERILIZATION

*The younger generation is the arrow,
the oldest is the arc.
(John Steinbeck)*

MEDICAL TRADING becomes GUARISCO MEDICAL STERILIZER s.r.l.

This year we have changed the name of our company with a new brand name that better represents our **identity as a manufacturer** of medical devices for sterilization.

Passion for work, for the customers and territory is the philosophy that has characterized the history of our company for more of 60 years. Values present in the DNA of GUARISCO MEDICAL since its origin, and carried on by three generations of Guarisco family, who have made GUARISCO MEDICAL synonymous of excellence and safety in the Italian and International market.

It all began in the early 1960s, thanks to the intuition of Pasquale Guarisco who developed the first machines for sterilization using high temperatures, thus giving life to the line of sterilizers: the **HOT DRY** series.

With the second generation and the development of the **HYDRA 100**, the first autoclave produced in Italy in the early 90s, GUARISCO MEDICAL has managed to establish itself on the international market by spreading the values of quality, reliability and Made in Italy.

It is thanks to these principles and to the experience earned over many years, that the third generation of GUARISCO MEDICAL has developed a new class B autoclave **PHOENIX BLU**

From autoclaves to thermo-sealer, all our products are designed and manufactured to ensure total safety on the products in the medical, dental and beauty market, with particular attention to all operators and patient.

Safety, Made in Italy quality and reliability are the values that for more than sixty years have guided the work of Medical Trading, and allowing all our products to meet the highest standards of the global market and legislation.

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QUALITY MANAGEMENT SYSTEM

To date, it can be said that the word AUTOCLAVE has become synonymous with SAFETY.

We at *MEDICAL TRADING* know this very well and that is why we are committed to getting all the certifications for our factory and our products.

In fact, today our company complies with the standards required by UNI EN ISO 13485:2004 for the manufacturing and servicing on medical devices in Class IIB.

Our products comply with the European Directives:

- Directive 2017/745-(MDR) for medical devices
- Directive EN13060 for small autoclaves.
- UNI EN ISO 13485:2016

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CLASS B

CLASS B - Big Small Sterilizers

Small autoclaves with similar performance to hospital autoclaves. With these devices it is possible to sterilize any type of load: solids, bodies Cables (Type A and Type B), porous materials, textile loads, wrapped and unwrapped instruments.

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Class N - Naked solid product

Small autoclaves suitable for the sterilization of **unwrapped solid instruments**. With these machines you can not sterilize hollow instruments, porous or wrapped. Although it has a limitation in the type of instruments to be sterilized, the autoclave in class N has many strengths, such as the **smaller dimensions**, the **rapidity of the cycle**, **reliability** and of course the **lower price**.

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CLASS N

CLASS S - Specified by the manufacturer

It is a middle class, whose type of instruments to be sterilized must be specified by the manufacturer. **You can NOT sterilize all types of instruments. Many security in less**, slightly lower price, and in some cases, higher price than the *phoenix Blu*.

CLASSE S - Specified by the manufacturer

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CLASS S

SOLID INSTRUMENT

Tools, which can be subjected to sterilization, without cavities and without obstacles for the penetration of steam.

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POROUS MATERIALS

Porous bodies are simple or compound materials that can absorb fluids (tissues, gowns, etc.)

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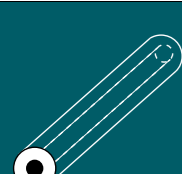
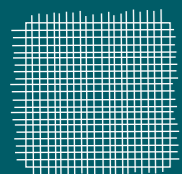
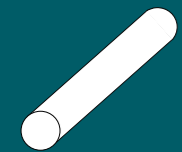
HOLLOW INSTRUMENT


The hollow material are materials or instrument with cavities, obstructions, etc ...
Type A: turbines, handpieces and devices with blind holes or small
Type B: cannulas, tubes or devices with large passages

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OUR PRODUCTION

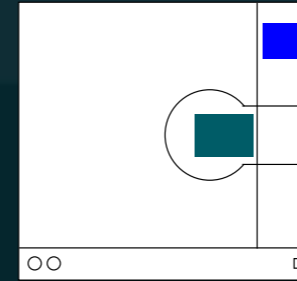CLASS B AUTOCLAVES

According to the Directive UNI EN 13060, to which all our autoclaves are comply, Class B autoclaves are the only ones that can sterilize any kind of instrument. Autoclaves are fully automatic and with similar performance to the hospital autoclaves.

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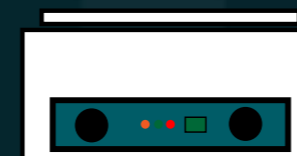
CLASS N AUTOCLAVES

The autoclave class N is suitable to sterilizing only unwrapped solid instruments. Although it has a limitation in the type of instruments to be sterilized, the class N autoclave has several strengths, such as the smaller size, a faster sterilization cycle, reliability and of course the price.

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
HOT AIR STERILIZER

The devices are hot air sterilizers that are commonly called "DRY HEATERS" and can be of two types, depending on the mechanism of air circulation inside the devices.

- 1) Natural air circulation
- 2) Forced air circulation

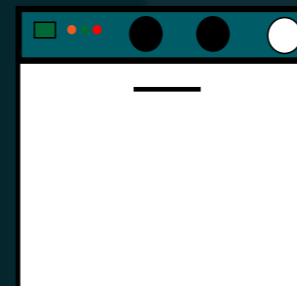The operation for both types, is based on heating, with electrical resistances, of the air. This circulates in the sterilization chamber and sterilizes the instruments at the required temperature.

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THERMO SEALER AND ACCESSORIES FOR STERILIZATION

The main products of our company are autoclaves and sterilizers. To offer a complete service to our customers, over the years we have enriched our production to a variety of accessories: Thermo-sealer, distiller water, trays, baskets and boxes, bags and rolls for sterilization, Control Test for autoclaves and sterilizers.

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phoenix Blu

phoenix Blu LCD

Un AUTOCLAVE per tutti!!
 Today the sterilization must be applied in different sectors and different places, sometimes even very contrasting between them: in the department of a large hospital, in a small dentist's office; from the classic dental practice to the side of a beautician's bed. For this reason we have developed a simple autoclave suitable for any use. Our cycles are not divided by type of instruments, but they are clear and simple. Just press a button is possible to customize each cycle. The Phoenix Blu autoclave is equipped with manual door closing, multi-language and graphic LCD display that show in real-time the parameter of the sterilization cycle. The basic version is also equipped with a USB output for saving data via PEN-DRIVE (included). Just connect the pendrive to the PC to show the sterilization data on the computer. The Phoenix Blue is available in special versions:

- *phoenix Blu LCD PRINTER*
- *phoenix Blu LCD PLUS**
- *phoenix Blu LCD AWF* (Automatic Water filling)

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phoenix Blu LED

phoenix Blu LED

This special version has been specially developed for particular markets where the basic data is only 3:

- QUALITY - SIMPLICITY - CONVENIENCE

We then tried to **SIMPLIFY the autoclave** by replacing the LCD screen with 3 LED screens that indicate the basic data of the autoclave: TEMPERATURE - PRESSURE - TIME.

The **QUALITY has not changed** than the other versions, in fact all the components are unchanged, so it is the **same features and specifics** of the LCD versions.

To stand out from the competition, which continues to raise prices, we lower it. In fact, this model has allowed us to further **reduce the price of our autoclave !!**

The Phoenix Blu LED is available also in the following special version

- **phoenix Blu LED PRINTER**
- **phoenix Blu LED PLUS+**
- **phoenix Blu LED AWF** (Automatic Water filling)

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SPECIAL VERSION

phoenix Blu LCD Printer

phoenix Blu LCD Printer is a special version of the autoclave phoenix Blu.

It is available with all our class B autoclave of any capacity (12-18-24l).
 It has the same technical features of the basic version but it is equipped with an integrated printer for automatic printing of the data, on thermal paper.
 In fact, with this version, at the end of each sterilization cycle it is possible to automatically print the data with all the parameters of the cycle performed
 The USB output is always present for the digital storage of sterilization data via PC.

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phoenix Blu LED Printer

phoenix Blu PRINTER is a special version of the autoclave phoenix Blu.

It is available with all our class B autoclave of any capacity (12-18-24l).
 It has the same technical features of the basic version but it is equipped with an integrated printer for automatic printing of the data, on thermal paper.
 In fact, with this version, at the end of each sterilization cycle it is possible to automatically print the data with all the parameters of the cycle performed
 The USB output is always present for the digital storage of sterilization data via PC.

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phoenix Blu plus⁺

THE AUTOCLAVE WITH SOMETHING +

phoenix Blu plus⁺
with a 3-stage osmosis system
to be placed on the side of the
autoclave or on a nearby wall

phoenix Blu TOP
with a in-line osmosis system
to place on the TOP of autoclave.

phoenix Blu AWF

The new autoclave *phoenix Blu plus⁺*, is fully automatic, the only thing left you need to do is only place the instruments inside the chamber.

Thanks to the new **WATER DEPURATOR BY REVERSE OSMOSIS SYSTEM**, you can directly connect the water supply to the autoclave. The water is automatically purified (<10 µs) and loaded into the autoclave tank. When the cycle is finished the water will be directly discharged.

The *phoenix Blu plus⁺*, is equipped with an integrated printer, which directly prints the data of the sterilization cycle. Normally the osmosis system is supplied separately from the autoclave, but with the predisposition for **anchoring on the right side of autoclave**. If you prefer the osmosis system can be place near the autoclave (up to 3m away) on a wall or furniture.

On request is available the trolley with autoclave for the placed of osmosis depurator.

For small studies or for sterilization cabinet it's available the *phoenix Blu TOP⁺* with horizontal water depurator that is possible to place on the top of the autoclaves.

All version are available with LED display.

- *phoenix Blu LCD plus⁺*
- *phoenix Blu LCD TOP*
- *phoenix Blu LED plus⁺*
- *phoenix Blu LED TOP*

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phoenix Blu AWF is a special version of the phoenix Blu autoclave. It is available for all our class B autoclaves of any capacity and has the same technical characteristics as the basic version, but equipped with an **AUTOMATIC WATER FILLING SYSTEM**.

Thanks to the additional **BUILT-IN PUMP**, it is possible to automatically load the water from an external tank placed under/near the autoclave.

This autoclave is also equipped to connecting the **REVERSE OSMOSIS PURIFIER** (to be purchased separately).

Available with LED display: *phoenix Blu LED AWF*

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Available with LED display: *phoenix Blu LED AWF*

CLASS B AUTOCLAVE

NEW PHOENIX B

This changes everything. Again.

Exactly 10 years have passed since our research center is developing and set to work to develop the class B autoclave, a work completed in 2011 with the presentation of the autoclave *phoenix Blu*

This year we have completely revolutionized our autoclave, with a **new and modern Design**. Our engineers, after several hours of study, have developed a design that blends modernity (of aeronautical inspiration) but still maintaining a distinctive trait of our company. Furthermore, an **automatic door closing system** has been implemented.

It is enough to lightly rest the door, a sensor will understand this movement and **automatically perform the mechanical closing**. In case of need it is always possible to open the door, through the emergency opening.

The new software that from 2020 will be present on all our class B autoclaves includes 14 sterilization cycles.

The new PHOENIX B will be available only with the LCD display and with an integrated printer.

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SAME FEATURES, LESS SPACE, LESS TIME

Following several requests coming mainly from the beauty market, our R&D department developed the autoclave *phoenix Blu 12* liter. **Same features, same components in a reduced space**. This autoclave has been developed specifically for the beauty market, but having the same functions as the "normal" version is also perfect for small medical/dental/veterinary study. The smaller dimensions is not the only advantage, because fewer size of the chamber involves less time of the filling phase and less time of the sterilization cycle. A **class B sterilization cycle** is completed in **just 15 minutes**. Available in the following version:

- *phoenix Blu 12* - *phoenix Blu 12 USB* (with USB output)
- *phoenix Blu PRINTER 12* (with USB output and integrated PRINTER)

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
TECHNICAL DATA

External Size	407 mm. L; 410 mm. H.; 570 mm. D
Size of sterilization chamber	ø210 x 360 mm.
Tray size	310 x 160 mm.
Capacity	12 Liters
Power Consumption	1800 W.
Tank Capacity	Distilled water: 3 l - Used water: 4,0 l
Weights	Net weightKg 42 - Gross weight Kg 44

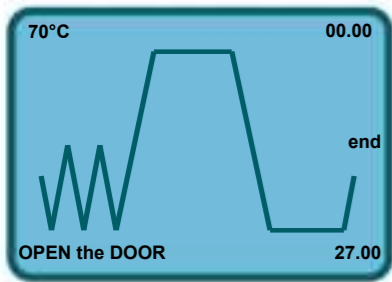phoenix Blu

B-CLASS

LCD DISPLAY - Multi-language and graphic display.

The display shows the graph of the cycle in use and provides the operator with instantaneous data on the pressure or temperature within the chamber or on the function in progress. The screen shows also warning/error messages along with an audible alarm

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FAST

With the standby mode, the sterilization chamber is heated so as to eliminate the operation of heating and save about 10-15 minutes on the next sterilization cycle.

DRY

You can choose the drying time between 1-5-10 minutes.

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12 STERILIZATION CYCLES

4 cycle at 134°C or 121°C in class B or class S. With the DRY button you can choose the time of the drying step (1-5-10 min) so as obtain up to 12 cycles. **ALL CYCLES CAN BE A FLASH CYCLE**

3 TEST CYCLES

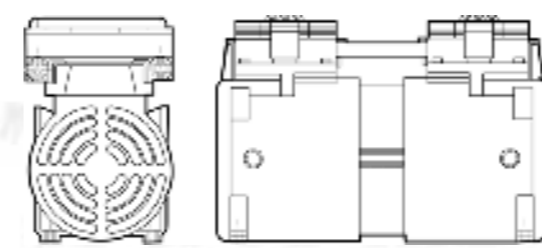
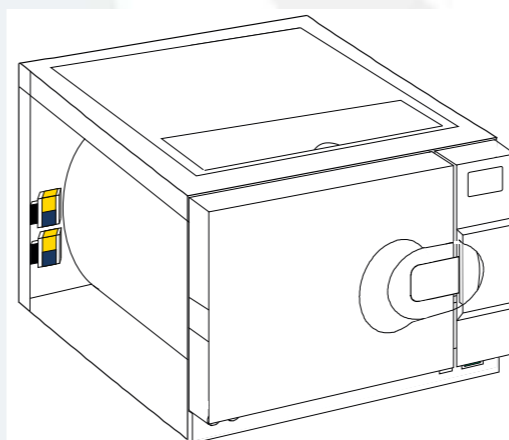
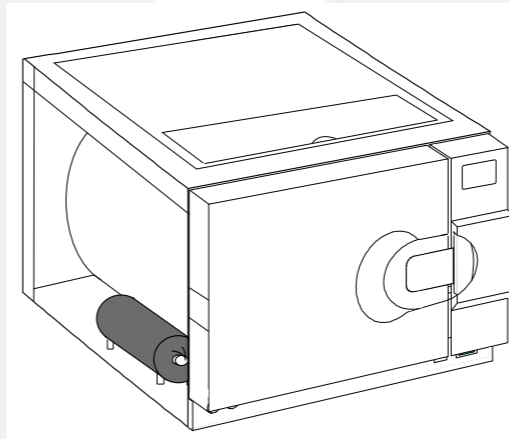
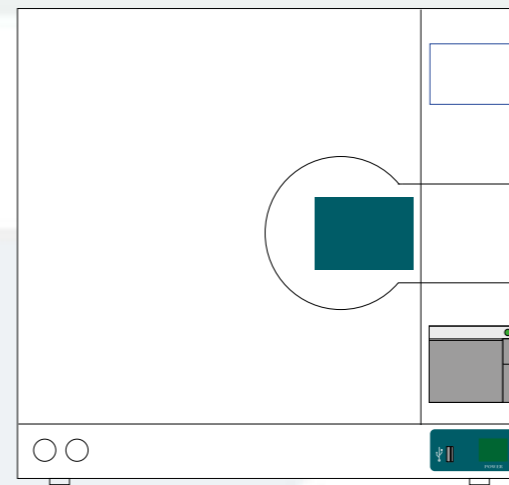
Helix Test, Bowie and Dick Test and Vacuum Test

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Helix Test, Bowie and Dick Test e Test del Vuoto

USB OUTPUT & INTEGRATED PRINTER

Just connect the flash-drive provided in the usb-output to save and store indefinitely the sterilization data. It's so possibile to read the sterilization data on the PC by the PEN-DRIVE included. On request you can have the integrated printer for printing the sterilization data on thermal paper. All our autoclave are equipped with RS232 output.

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ULTRA RAPID STEAM GENERATOR

The new Phoenix Blu are equipped with a last generation and **ULTRA-RAPID**, external steam generator. This feature, has the benefit to decrease the filling times of the sterilization chamber and the time of all cycle, but especially this feature transform the water into steam in a very short time and avoid the deposit of limestone inside. With the **GENERATOR CLEANING CYCLE** it is possible to remove any impurities and thus increase the useful life of this fundamental component.

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QUALITY AND RAPID

The main features required by each user are the **QUALITY** and **RAPID**. The new Phoenix Blue has been designed to meet these two features completely. In fact, in our autoclaves we can only find high quality components. But not only, all the autoclave and positioning of each single component has been carefully studied to minimize ordinary and extraordinary maintenance times.

DOUBLE HEAD VACUUM PUMP

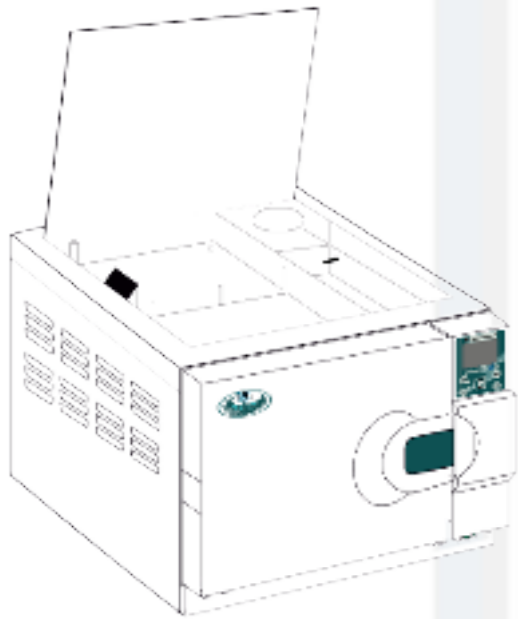
The vacuum pump is the most important element of the entire autoclave, as it adjusts the function of the vacuum, the major step for a sterilization cycle in class B. On our autoclaves we assemble only high-efficiency pumps with double head. **(3 Fractionated Vacuum)**.

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INDEPENDENT AND INSPECTABLE TANKS

The two separate tanks for distilled water and used water can easily be inspected through the top cover, which is secured by simple screws. Only unscrew them to access tanks so you can carry out normal maintenance tasks very quickly. The autoclave is also equipped with a **WATER QUALITY CONTROL SYSTEM** that check the conductivity of the distilled water and show error messages is it over 15µs.

AIR/WATER SEPARATOR

The autoclave is equipped with 2 capacitors: The first is to condense the steam into water, the second has the function of **separating the water from air**. This function is very important, because it prevents that the steam and/or further liquids (corrosive) do not go inside the pipes and prevent the breakage of components such as the vacuum pump.

RAPID DRAIN PIPE

All our autoclaves are equipped with rapid drain system for the loading and unloading of the water, positioned on the front of the sterilizer makes the operations easy and fast. On request you can have a rear outlet, already standard on some models, with the ability to connect directly to the drain water of the study.

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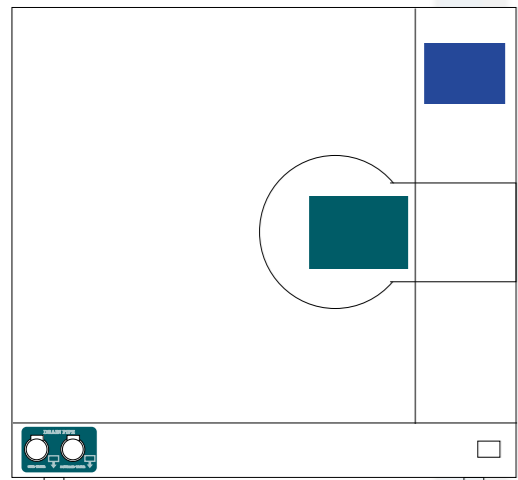
MODULAR BASKET TRAYS & POUCH RACK

All autoclave are equipped with a modular basket trays. This particular basket can support up to 5 trays, or you just rotate it to bring up to 3 cassettes. (3 trays included in the price). On request there is also available a POUCHES RACK. This allows you to place various wrapped tools, saving you a lot of time and money.

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Model Name	CLASS B	LCD DISPLAY	LED DISPLAY	STEAM GENERATOR	2 WATER TANK	VACUUM PUMP 3 fractioned vacuum	1 BASKET TRAYS	3 TRAYS	5 TRAYS	CLASS B	USB OUTPUT	INTEGRATED PRINTER	BACK DRAIN PIPE	AUTOMATIC WATER FILL.	AUTOMATIC WATER FILL. BY PUMP	OSMOSIS DEPUP. VERTICAL SYSTEM	OSMOSIS DEPUPATOR HORIZONTAL SYSTEM	
<i>phoenix Blu LED TOP</i> 18 - 24	●	X	●	●	●	●	●	●	□	●	●	●	●	●	●	X	X	●
<i>phoenix Blu LED PLUS</i> 18 - 24	●	X	●	●	●	●	●	●	□	●	●	●	●	●	●	X	●	X
<i>phoenix Blu LED AWF</i> 18 - 24	●	X	●	●	●	●	●	●	□	●	●	●	●	●	●	●	X	X
<i>phoenix Blu LED PRINTER</i> 18 - 24	●	X	●	●	●	●	●	●	□	●	●	●	●	X	X	X	X	X
<i>phoenix Blu LED</i> 18 - 24	●	X	●	●	●	●	●	●	□	●	●	□	□	X	X	X	X	X
<i>phoenix Blu TOP</i> 12 - 18 - 24	●	●	X	●	●	●	●	●	□	●	●	●	●	●	●	X	X	●
<i>phoenix Blu PLUS</i> 12 - 18 - 24	●	●	X	●	●	●	●	●	□	●	●	●	●	●	X	●	X	X
<i>phoenix Blu AWF</i> 18 - 24	●	●	X	●	●	●	●	●	□	●	●	●	●	●	●	●	X	X
<i>phoenix Blu PRINTER</i> 12 - 18 - 24	●	●	X	●	●	●	●	●	□	●	●	●	●	X	X	X	X	X
<i>phoenix Blu</i> 12 - 18 - 24	●	●	X	●	●	●	●	●	□	●	●	□	□	X	X	X	X	X

● INCLUDED AS STANDARD


□ OPTIONAL ON REQUEST

X NOT AVAILABLE

TECHNICAL DATA

DATI TECNICI

Model Name	<i>phoenix Blu</i> 18 L	<i>phoenix Blu</i> 24 L
Dimension and volume	462 x 445 x 600 mm (w x h x d)	
Weights (machine empty with basket)	Net weight: Kg 54 Gross weight: Kg 56	Net weight: Kg 56 Gross weight: Kg 59
Minimum depth of the support surface	500 mm	
Minimum width of the support surface	460 mm	
Minimum space for water loading	165 mm	
Weight on support area (tank full and chamber with max. load)	Load on each foot: 17,25 kg	
Voltage	230 V - 50/60 Hz	
Fuses	2 x 12FAL 250V.	
Power Consumption	1800 W	
Sterilization temperature	121°C - 134°C	
Working pressure	Cycle 121°C: 1,0 - 1,3 bar (STP) Cycle 134°C: 2,0 - 2,4 bar (STP)	
MAX working empty	-0,9 bar (STP)	
MAX pressure	2,5 bar (STP)	
Sterilization chamber	255 X 360 mm (ø x d)	255 x 460 mm. (ø x d)
	STAINLESS STEEL - AISI 316	
Size of trays (3 trays included)	285 x 190 mm.	385 x 190 mm.
Water tank	Distilled water 4,6 litres Used water:5,0 litres ABS - Anticorrosive	
Water consumption per cycle	0,15 / 0,30 l - 15 cycles	0,16 / 0,40 l - 11 cycles
Bacteriological filter	Diameter: ø 56 mm - Filtering capacity: 0.3 µm	
Operating range	Indoor use, maximum relative humidity 85% Maximum line-voltage deviation ±10% Installation category II, Pollution level 2	
Max noise level	50 dB	
Operating Altitude	<2000m (IF>1000 make a Parameter setting)	

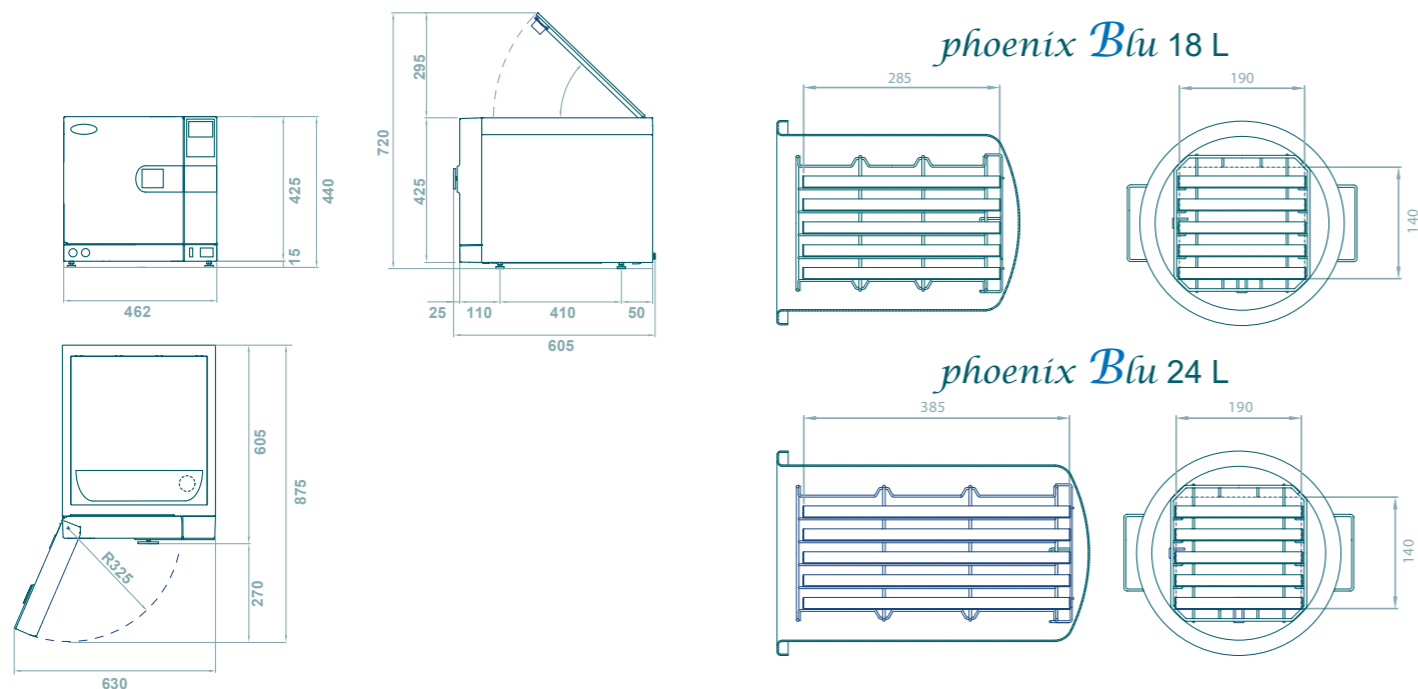TECHNICAL DATA

DATI TECNICI

SECURITY DEVICE	COMPLIANCE TO EUROPEAN DIRECTIVES	CERTIFICATION
<ul style="list-style-type: none"> - Safety valve certified TÜV - PED IV - Overpressure control - Safety thermostat - Safety door lock (3 devices: mechanical, dynamic, electrical) - Microprocessor - Real-time self-diagnosis system - Electronic temperature and pressure control 	<ul style="list-style-type: none"> - EN 13060 - EN ISO 14971 - EN 61010-1 - EN 61010-2-040 - EN 61326 - EN 13445 	<ul style="list-style-type: none"> - The autoclave complies with Directive 93/42-ECC and addendum. - Class IIb device - The safety valve and Chamber complies with PED 97/23/ECC - Certified company UNI EN ISO 134885

Autoclave Equipment:

- User manual
- 1 USB pendrive(2Gb)
- 1 handle for trays
- Technical assistance manual (on request)
- Declaration of conformity
- 1 modular basket trays
- 1 Power cord
- Declaration of conformity for the safety valve
- 3 trays
- 1 water drain tube with quick connector

Maintenance equipment:

- 1 door seal
- 1 bacteriological filter (air)
- 1 steam discharge filter
- 2 power fuses
- 1 distilled water tank filter
- 1 fuse for pc board

TIME AND OPERATION OF THE PROGRAMS

Time of the Cycle	Universal Cycle	Prione Cycle	Unwrapped Cycle	Rubber & Plastic Cycle	Helix Test B&D Test	Leak Test	Cleaning Generator
Sterilization Temperature	134 °C	134 °C	134 °C	121 °C	134 °C	==	==
Sterilization Pressure (bar)	2,10	2,10	2,10	1,10	2,10	==	==
Class (EN13060)	B 3 fractioned	B 3 fractioned	S 1 vacuum	B 3 fractioned	B 3 fractioned	==	==
Time of Void	4 min.	4 min.	2 min.	4 min.	4 min.	4 min.	20 min.
Steam InjectionTime	6 min.	6 min.	6 min.	6 min.	6 min.	==	==
Sterilization Time	4 min.	18 min.	4. min.	20 min.	3,5 min.	10 min.	==
Drying Time	1-5-10 min.	1-5-10 min	1-5-10 min	1-5-10 min	1 min.	==	10 min
Total Time	19 min*	28 min*	17 min*	35 min*	14,5 min.	14 min.	30 min.

MAX LOAD	<i>phoenix Blu</i> 18 l.	<i>phoenix Blu</i> 24 l.
UNIVERSAL CYCLE / PRIONE CYCLE WRAPPED SOLID INSTRUMENT	max 2,5 Kg.	max 3,5 Kg.
UNIVERSAL CYCLE / PRIONE CYCLE HOLE BODY	max 2,5 Kg.	max 3,5 Kg.
UNIVERSAL CYCLE / PRIONE CYCLE POROUS MATERIAL	max 1,5 Kg.	max 2 Kg.
UNWRAPPED CYCLE UNWRAPPED INSTRUMENT	max 5 Kg.	max 7 Kg.
PLASTIC MATERIAL CYCLE WRAPPED INSTRUMENT	max 2,5 Kg.	max 3,5 Kg.

*Time does not include pre-heating time.
It usually ranges between 10-15 minutes, depending on the load.
Total time is calculated with quick-drying of 5 min on 18 l version.

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CLASS N AUTOCLAVE

HYDRA 100

**CLASS N AUTOCLAVE
COMPLY TO DIRECTIVE 93/42
MEDICAL DEVICES CLASS IIb**

A HYDRA 100 IS FOREVER

The autoclave HYDRA 100 is equipped by a heating element placed under the sterilization chamber which allows the heating to 125 ° C of the distilled water. The pressure that is created inside the sterilization chamber, in combination with the temperature, allow the complete asepsis on the instruments. The sterilizer is equipped with several sensors that monitor the reaching of the temperature, which is preset and can not be changed to prevent tampering.

The operation is very easy just rotate the knob that controls the time of the sterilization cycle and the autoclave begins the heating phase. Upon reaching 125 ° C a light signal will inform the start of the sterilization phase.

At the end of the cycle just open the vent valve and wait 2 minutes. The sterilization cycle is now complete and it is possible to unload the sterilized instruments.

The machine is equipped with several safety systems, in particular with a thermostat that, in the event of an excessive temperature, stops the sterilization cycle. It is also equipped with a safety valve with TUV certification (PED IV), the same installed on the class B autoclaves.

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HYDRA 100 AUTOMATIC

THE MAIN QUALITY IS RELIABILITY!
Autoclave Hydra 100 is a Class N autoclave, is compliant with Directive 93/42/EEC on small steam sterilizers. With its reliability, speed, and less space is a good alternative to class B autoclaves for the sterilization of unwrapped solid instruments.

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SPECIFICATIONS	
External size	ø 230 – H 370 mm.
Internal size	ø 180 – H 200 mm.
Capacity	4 L
Production of distilled water	1,5 L / h
Control of temperature	Electronic
Absorbed power	750 Watt
Voltage	230/240 V.
Weight	Net Kg. 3,5. Gross Kg. 4,5

The distiller ensures distilled water of excellent quality, thanks to a boiling procedure with forced condensation. Thanks to a high voltage resistance produces 4 liters of excellent distilled water in about 2 hours. The use will avoid the purchase and storage of bulky drums of distilled water, with a considerable saving of money and energy.

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The ultrasonic cleaning is a modern, safe and rapid system to obtain in a short time a guarantee of deep cleaning. The main advantage consists in a perfect cleaning of the objects through the elimination of dirt, encrustations and contaminants among the most stubborn arranged in cavities that are difficult to reach manually.

The new ultrasonic cleaner **CYGNUS** is equipped with DEGASS and SWEEP technology SYSTEM which contain the best that can be requested from an ultrasonic cleaning device.

Main features:

- Capacity: 6 liters.
- Instrument basket and lid included
- Electronic temperature selection control.
- Sweep zone control for a better distribution of cavitation over the whole tank.
- Degas command, for eliminating the gas present in the tank water.
- Liquid discharge with integrated valve.
- Electronic time selection control.

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STERILIZATION CONTROL TEST

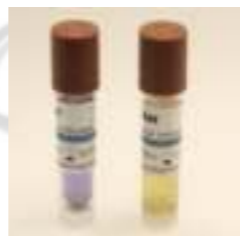
To try to give you a full service, our company can provide you with everything you need to carry out sterilization checks on our machines. The following types of test that are available are:

- HELIX TEST
- B&D TEST
- TST TEST and PRIONE TEST class 6 or class 5
- BIOLOGICAL INDICATORS
- HOT AIR STERILIZER CONTROL TEST

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
External size	L 330 – H 310 – W 180 mm.
Size of washing chamber	L 300 – H 150 – W 155 mm.
Sterilizing liquid	Distilled water
Absorbed power	150 Watt
Absorbed power with heating	300 Watt
Voltage	220/230 V.
Capacity	6 Liters
Cleaning temperature	0-80 °C
Cleaning time	1 - 99 min
Ultrasound frequency	40 khz ± 5%
Number of transducers	2
Weight	Net 4 kg. – Gross 5 kg.

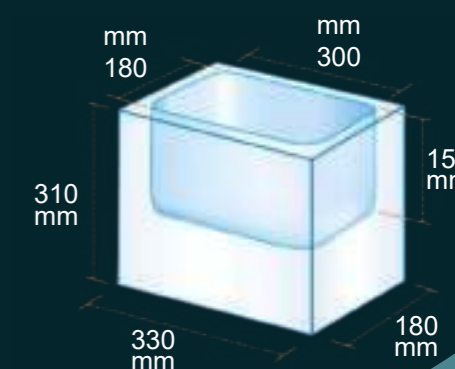SEALER SIGILLATRICE

Sealer SAGITTARIUM

Sagittarium is the new electronic thermosealer, a product of design, project to reduce the time of sealing and to monitor the quality of the weld. This new model is equipped with electronic card for the control and guarantee of maintenance of sterilization. The thermosealer Sagittarium are equipped with an electronic control of sealing parameters (time and temperature) and guarantee a sealing band of 12 mm, well exceeding the standards required by current regulations. The sound and light signals also provide information on the correct operating conditions of the heat sealer. Compact size, comfort, practicality and functionality come together in Sagittarium: if you choose to disconnect the roll holder, the machine takes up only 30 cm deep and can also be placed on small planes and shelves. The shell is produced in new resistant and recyclable material, it has an innovative design and is made with advanced materials and light.

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TECHNICAL DATA	
Type of seal:	Constant heat
Welding height:	12 mm
Length of sealing bar:	320 mm
Roll holder width:	320 mm.
Cutting system:	Incorporated
Temperature	200±10 °C (Automatic control)
Warm-up time	3 minutes
Sealing time	3 seconds (Can be increased as needed)
Sealing information completed	Light and sound alerts
Dimension: L x H x W	365x245x310 mm.
Weight	Net 4,8 kg. - Gross 7,0 kg
Power	500 Watt
Voltage	230/240 V.
Frequency	50/60 Hz
Structure material	Steel / ABS injection

ROTARY SEALER SIGILLATRICE ROTATIVA

TAURUS rotary sealer

TAURUS is new rotary sealer with printing function and touchscreen display. The use of this machine is simple and rational, just insert the bag with the tool on the special support of the sealer and the photocell automatically drags the envelope autonomously, sealing it and imprinting the packaging data. Using the comfortable and useful touch screen it is possible to adjust the usage parameters. It is also possible to change the printing information: Time and Date, Expired date, Batch number, Operator name, Autoclave identification number, Type of instruments, Company name, Sealer serial number, Total sealing counter, CE marks.

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TECHNICAL DATA	
Type of seal:	Rotary sealer
Display	7" TFT graphic colour - Touchscreen
Printout	24-dot matrix, double lined, for special characters
Welding height:	12 mm
Sealed edge infinitely adjustable	0 - 40 mm
Sealing seam distance from pack content	>30 mm (DIN 58953-7:2003)
Sealing temperature max.	220 °C ± 1% (tolerance monitored)
Sealing speed	10 m/Min (monitored)
Contact pressure	80 - 120 N (monitored)
Warmup Time	40 seconds
Sealing information completed	Light and sound alerts
Dimension: LxWx H	560x260x220 mm.
Weight	Net 18 kg. - Gross 20,0 kg
Power	500 Watt
Voltage	230/240 V.
Frequency	50/60 Hz
Fuse	5A x 2

RELLS AND POUCHES FOR STERILIZATION

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Our products for STEAM, EO and FORM sterilization are made by a plastic medical use material called laminated transparent film coming from "corona treatment" polyester and peelable shutterless polypropylene and by kraft paper certified and studied to assure an high antibacterial protection for STEAM, EO and FORM medical sterilization. The inks we use are certified ISO 11140, water based and non toxic with a clear change in color. Medical Device Class I

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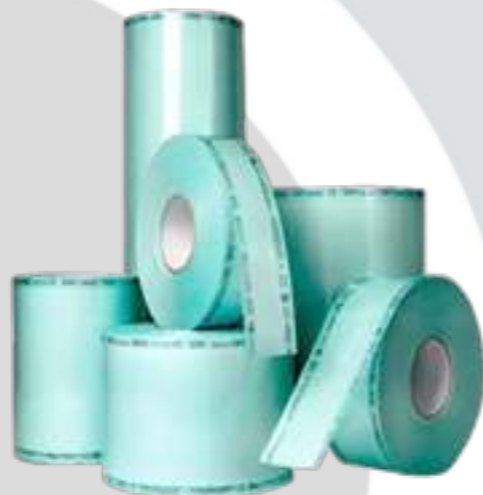
The sterilizers of HOT DRY series use the principle of heat transfer. An armored electrical resistance allows the air, within the sterilization chamber, to reach a uniform temperature (that can range from 160 to 180°C), and comes into contact with tools allows a complete and effective sterilization.

The main feature of the HOT DRY is the reliability. The external walls are made entirely of steel, painted with epoxy powder, resistant to high temperatures and resistant over time. While the inner walls of the sterilization chamber are made of stainless steel 18/8 polished. A slab of glass fiber insulating ensures the insulation of the heat, keeping the temperature constant inside, and the external walls to ambient temperature, ensuring safety for the operator. The sterilization time depending on the set temperature for a temperature of 180 °C, the sterilization time is 90 minutes. The line HOT DRY is suitable to sterilize all instruments resistant to high temperatures.

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HOT DRY 22 - 30 L.

The sterilizers of HOT DRY series use the principle of heat transfer. An armored electrical resistance allows the air, within the sterilization chamber, to reach a uniform temperature (that can range from 160 to 180°C), and comes into contact with tools allows a complete and effective sterilization.

The main feature of the HOT DRY is the reliability. The external walls are made entirely of steel, painted with epoxy powder, resistant to high temperatures and resistant over time. While the inner walls of the sterilization chamber are made of stainless steel 18/8 polished. A slab of glass fiber insulating ensures the insulation of the heat, keeping the temperature constant inside, and the external walls to ambient temperature, ensuring safety for the operator.

To achieve a better and uniform temperature in all the sterilization chamber the sterilizers of 22 or 30 liters are equipped with forced air circulation by fans. The fans in fact allow the air, contained in the sterilization chamber to reach the desired temperature in a uniform way, thus ensuring a complete asepsis of all the instruments.

The line HOT DRY is suitable to sterilize all instruments resistant to high temperatures.

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